

# Please Read Before Operating!


Your ActiveCare product is packaged with a packet of tools you will need for initial setup. Please locate this packet first.

## Quick Start Guide Medalist, Catalina & Renegade Power Wheelchairs


# ActiveCare

Medical


- 1.** Lift the cover shroud upward to remove it. On the CATALINA and MEDALIST, the two knobs behind the seat post must be loosened and removed before the shroud can be removed.

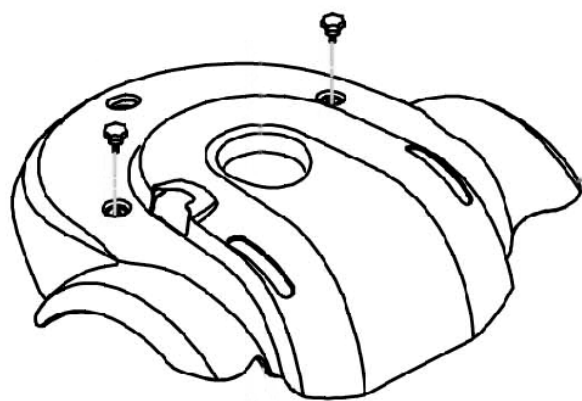

- 2.** Install the seat post. Ensure that the seat post is aligned so that the two flanges with holes are facing the rear of the product. Having these flanges rotated into any other position will result in instability in the seat.

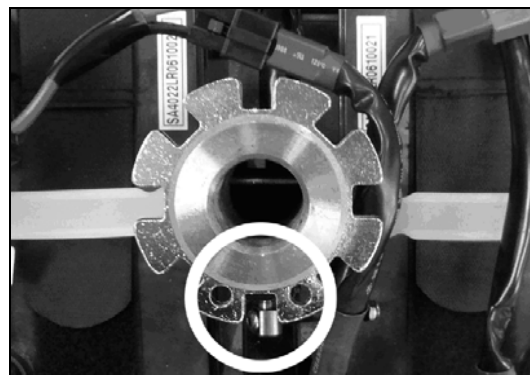

- 3.** From behind the product, looking forward, insert the seat bolt into hole on the left side of the seat post. This bolt is also used to adjust the seat height. Insert the bolt as far into the hole as it will go, making sure that the bolt passes all the way through to the hole on the opposite side. Install the washer and nut and tighten them using the included flat wrench.

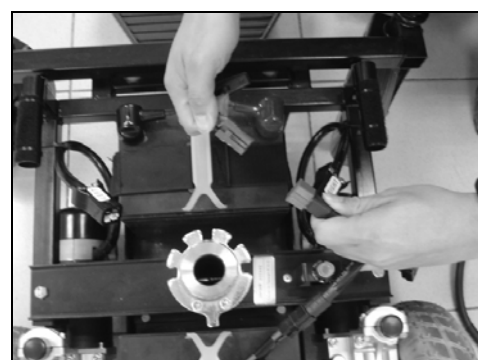


- 4.** Connect both battery connectors to the main cable.



- 6.** Connect the cable from the wheelchair's controller to the connection point in the wheelchair's cover shroud.



- 5.** Replace the product's cover shroud. The shroud should slide over the brake levers, which should be positioned in engaged mode. On the CATALINA and MEDALIST, replace the two knobs removed in Step 1.

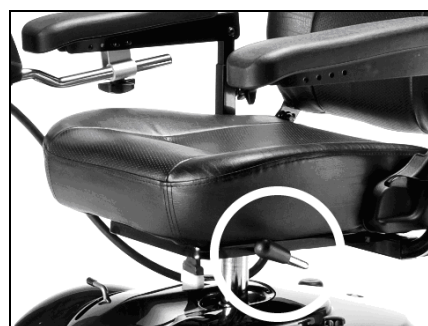

- 7.** Attach the seat by lifting up on the handle on the left side of the seat and placing the seat down on the wheelchair's seat post.




Fig. 1



Fig. 2

- 8.** The product's armrests come packaged as seen in Fig. 1. and must be rotated to appear as seen in Fig. 2. Remove the star-shaped knobs from the elbow tubes. With the star knobs removed, separate each assembly into 2 pieces. Reinsert the armrests so that they face forward and retighten the star knobs to lock them in place.

# Medalist, Catalina & Renegade Quick Start Guide

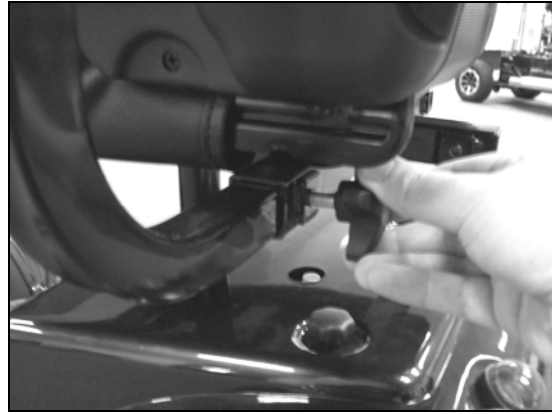

**9.** Insert the armrests into the product's seat frame. Your product may have either a star-shaped knob or an Allen bolt in the location. Tighten the knobs or bolts to lock the armrests in place.


Affix the headrest to the top of the seat by inserting the headrest posts into the holes in the top of the seat.

**10.** Insert the controller tube into the armrest tube. Secure the controller in place by tightening the knob on the underside of the armrest tube.



**11.** Fully charge the wheelchair's battery by inserting the round charger connector into the battery charging terminal on the wheelchair's control panel and inserting the 3-pronged plug into a standard electrical outlet.

Ensure that the 115V/230V switch on the product's battery charger is set to 115V. If the switch is set to 230V, the wheelchair will not charge properly.

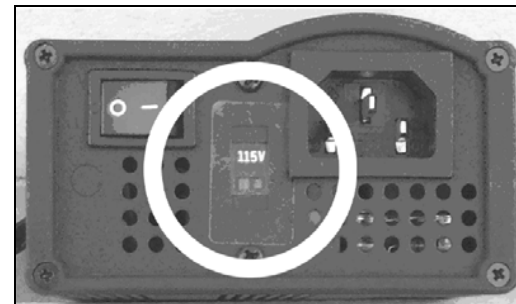

Full charging normally takes 6-8 hours, but can take as many as 10 hours depending on the batteries' charge level when starting and the ambient temperature.



## READ BEFORE UNPACKING

### Apparent Loss or Damage

Upon receipt of this shipment, if any evidence of loss or damage is revealed through visual inspection, then it must be noted on the freight bill and signed by the carrier's agent. Failure to do this may result in the carrier's refusal to honor loss or damage claim. The carrier will furnish you with the necessary forms to file the claim.

### Concealed Loss or Damage

When loss or damage is not apparent until the unit is unpacked, a claim for concealed loss or damage is required. When damage is discovered, make a written statement to the carrier for inspection. This request must be made within 15 days of delivery. Keep all cartons and fillers for inspection. Carrier will furnish inspection report and necessary forms to file concealed loss or damage claim, since such damage or loss is the carrier's responsibility.

**This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.**

**NOTE:** The illustrations provided may not reflect your actual unit.

[www.activecaremed.com](http://www.activecaremed.com)