




# Medalist, Catalina & Renegade Quick Start Guide

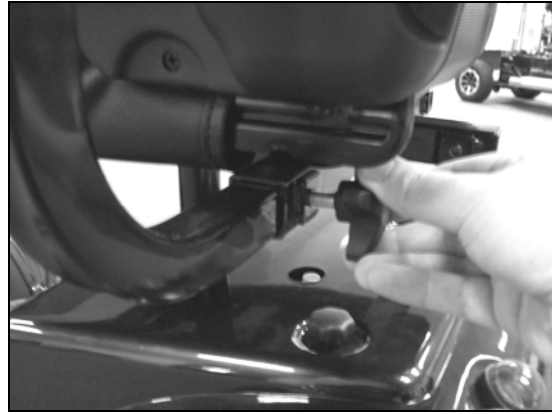

**9.** Insert the armrests into the product's seat frame. Your product may have either a star-shaped knob or an Allen bolt in the location. Tighten the knobs or bolts to lock the armrests in place.


Affix the headrest to the top of the seat by inserting the headrest posts into the holes in the top of the seat.

**10.** Insert the controller tube into the armrest tube. Secure the controller in place by tightening the knob on the underside of the armrest tube.



**11.** Fully charge the wheelchair's battery by inserting the round charger connector into the battery charging terminal on the wheelchair's control panel and inserting the 3-pronged plug into a standard electrical outlet.

Ensure that the 115V/230V switch on the product's battery charger is set to 115V. If the switch is set to 230V, the wheelchair will not charge properly.

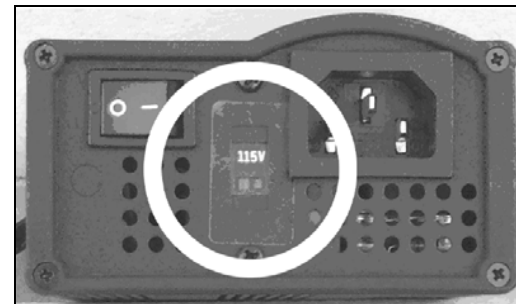

Full charging normally takes 6-8 hours, but can take as many as 10 hours depending on the batteries' charge level when starting and the ambient temperature.



## READ BEFORE UNPACKING

### Apparent Loss or Damage

Upon receipt of this shipment, if any evidence of loss or damage is revealed through visual inspection, then it must be noted on the freight bill and signed by the carrier's agent. Failure to do this may result in the carrier's refusal to honor loss or damage claim. The carrier will furnish you with the necessary forms to file the claim.

### Concealed Loss or Damage

When loss or damage is not apparent until the unit is unpacked, a claim for concealed loss or damage is required. When damage is discovered, make a written statement to the carrier for inspection. This request must be made within 15 days of delivery. Keep all cartons and fillers for inspection. Carrier will furnish inspection report and necessary forms to file concealed loss or damage claim, since such damage or loss is the carrier's responsibility.

**This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.**

**NOTE:** The illustrations provided may not reflect your actual unit.

[www.activecaremed.com](http://www.activecaremed.com)