

Please Read Before Operating!

Your ActiveCare product is packaged with a packet of tools you will need for initial setup. Please locate this packet first.

Quick Start Guide Wildcat Power Wheelchair

ActiveCare

Medical

1. Press the seat tubes down with both hands and make sure the crossbar tubes are tight and locked.



2. The backrest folding mechanism can fold the backrest.



3. Attach the Velcro strip on the bottom of the backrest to the Velcro strip on the underside of the seat cushion.



4. Place the two batteries into the frame under the seat.



5. Connect the two batteries together using the red battery connectors.



6. Assemble the footrest on the wheelchair and lock it by latch.



7. Fix the heel loop above the footrest.




8. Connect the three main cables to the controller: left motor connector to the left motor, right motor connector to the right motor, power connector to battery connector. Each connector is labeled with the correct connection to make.



9. Ensure that the freewheel mode levers near the rear wheels are set in "Drive" mode, as pictured.



Wildcat Quick Start Guide

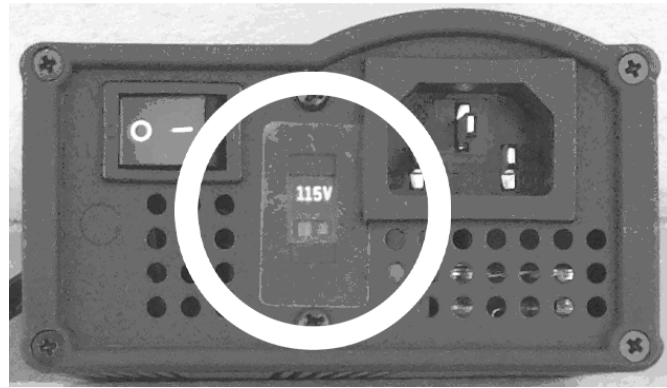

10. Ensure that the 115V/230V switch on the product's battery charger is set to 115V. If the switch is set to 230V, the scooter will not charge properly.

11. Fully charge the scooter's battery by inserting the round charger connector into the battery charging terminal on the front of the wheelchair's controller and inserting the 3-pronged plug into a standard electrical outlet.

Full charging normally takes 6-8 hours, but can take as many as 10 hours depending on the batteries' charge level when starting and the ambient temperature.



READ BEFORE UNPACKING

Apparent Loss or Damage

Upon receipt of this shipment, if any evidence of loss or damage is revealed through visual inspection, then it must be noted on the freight bill and signed by the carrier's agent. Failure to do this may result in the carrier's refusal to honor loss or damage claim. The carrier will furnish you with the necessary forms to file the claim.

Concealed Loss or Damage

When loss or damage is not apparent until the unit is unpacked, a claim for concealed loss or damage is required. When damage is discovered, make a written statement to the carrier for inspection. This request must be made within 15 days of delivery. Keep all cartons and fillers for inspection. Carrier will furnish inspection report and necessary forms to file concealed loss or damage claim, since such damage or loss is the carrier's responsibility.

This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.

NOTE: The illustrations provided may not reflect your actual unit.

www.activecaremed.com

